

KaVo QUATROcare® 2104 und KaVo QUATROcare® 2104 A

# It's in your hand.


Every KaVo instrument is top quality. And it will stay that way. To keep my KaVo instruments from losing their precision and value, I care for them with the KaVo QUATROcare®. At the same time, I create the ideal preconditions for additional hygiene. You can only save costs and achieve perfect results with perfectly cleaned and hygienically prepared instruments.

The KaVo QUATROcare® comes in two designs. As a standard unit, or with the innovative purging function. Compressed air additionally blows through the spray channels to remove any remaining oil residue and thoroughly clean the KaVo instruments.

### Advantages at a glance:

- 1 minute total cleaning cycle for 4 instruments
- Program-controlled automatic care cycle
- Perfect interior cleaning from both the aerosol foaming/ expansion effect and of course the aerosol's propellent pressure
- Optimum lubrication with automated dosing
- Easy operation with the push of a button prevents errors
- The spray channels are blown clean with the purging function\*
- Elimination of excess oil useage, prevents costly wastage and unnecessary mess
- Basic setup includes 4 couplings (1 x MULTIflex/3 x INTRAmatic)
- Universally applicable (Adapters for all conventional makes)
- Environmentally-friendly KaVo QUATROcare® spray
- Modern and ergonomic design

\*only available with the KaVo QUATROcare® 2104 A

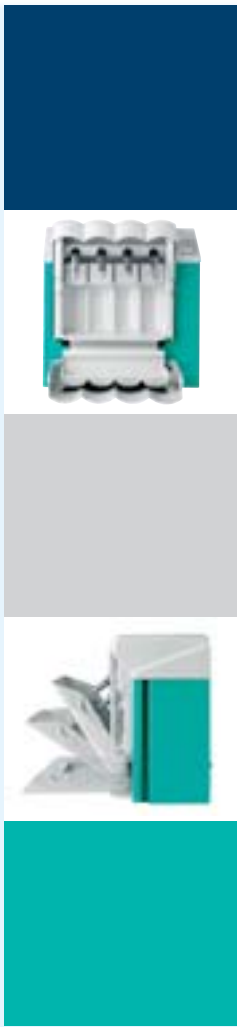

*KaVo QUATROcare® 2104.  
Device in the standard design without compressed air connection*  
*KaVo QUATROcare® 2104 A.  
Device with compressed air connection to blow clean the smallest spray channel*

Material-Nr.: 1.004.1434.08/06 en We reserve the right to make technical modifications. Slight colour differences are due to the printing process. © Copyright KaVo Dental GmbH.

KaVo QUATROcare®

KaVo QUATROcare®

# KaVo QUATROcare®. Perfect care.



KaVo. Dental Excellence.

KaVo Dental GmbH · D-88400 Biberach/Riß · Telefon +49 7351 56-0 · Fax +49 7351 56-1488 · www.kavo.com



KaVo. Dental Excellence.

KaVo QUATTROcare®

# Maximum efficiency.

A long service life is a question of cleaning and care and not just quality. With the KaVo QUATTROcare®, KaVo has set another international benchmark for efficiency in instrument care. The innovative KaVo QUATTROcare® system that has been proven for years is incomparably thorough, economical and easy to use. Perfect for all my instruments that are used daily in my practice.

"Dental Excellence" led you to select KaVo turbines, handpieces and contra-angle handpieces. The KaVo QUATTROcare® is the logical extension of this ideology. To your great benefit. The internationally established KaVo care system is optimised for the daily requirements of your practice, and joins the latest care technology with highly modern design and maximum efficiency. KaVo QUATTROcare® maintains the unique quality of the instruments, extends their life and establishes the prerequisites for meeting stringent hygiene regulations.



KaVo QUATTROcare®

# Easy to use.

Given the daily tasks of my practice, there is little time left to care for my high quality KaVo instruments. There is not much time left for the office team either. It is therefore, all the more important to rely on a well-conceived care system that anyone can operate. Insert the instruments. Close the front door. Start. Finished. Four at once. Just like that. Excellent.

Just a press of the button. Fully automated. The KaVo QUATTROcare® offers maximum speed and user-friendliness. The KaVo QUATTROcare® is so easy to operate. Simultaneously insert four instruments on the four couplings, close the door and start the care program. A complete care cycle lasts only 60 seconds. The KaVo QUATTROcare® is immediately ready to use.



- 1 Insert the instruments.  
To insert the instruments, a MULTiflex coupling is available as well as three INTRAMatic couplings.
- 2 Close the door.  
And you're ready to go.
- 3 Press the start button  
Fully automated cleaning and care of up to four instruments in just 60 seconds.
- 4 Finished.  
The cleaned instruments are ready to be sterilised.


KaVo QUATTROcare®

# Optimum cleaning.

The difference is in the details. KaVo QUATTROcare® cleans even the smallest niche without residue. The speed of the KaVo QUATTROcare® lessens the workload in my practice and gives me the assurance of optimum cleaning and perfect care. The service life of my instruments is extended.

Your instruments are ready to be used immediately after cleaning and care. Just like new. KaVo QUATTROcare® cleans by foaming the cleaning components. In addition, the automated dosing system ensures optimum lubrication of the turbines, handpieces and contra-angle handpieces.

The KaVo QUATTROcare® was developed based on the most current quality and safety standards. And it is universal. That means, you can clean your high-quality KaVo instruments using the KaVo QUATTROcare® as well as instruments from other manufacturers.

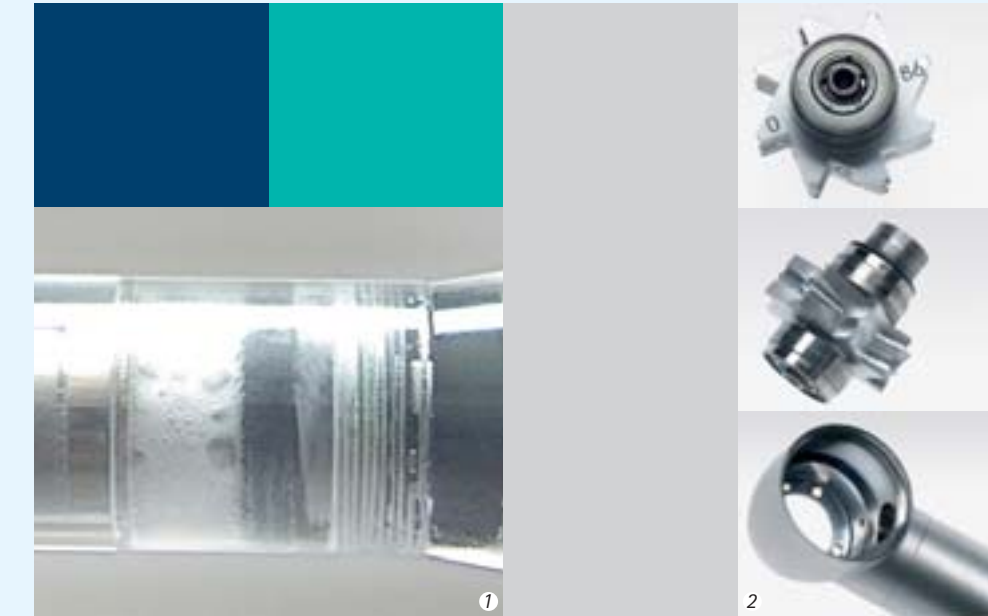

- 1 The KaVo QUATTROcare® effect.  
The forceful expansion of the QUATTROcare® cleaning agent inside the instruments dissolves and removes all residue.
- 2 Efficiency in the dental practice.  
KaVo SUPERtorque LUX 650 B, cleaned with the KaVo QUATTROcare®. Seven years in daily use without a breakdown or exchanging the rotor. Always optimally clean and maintained, always ready to use.