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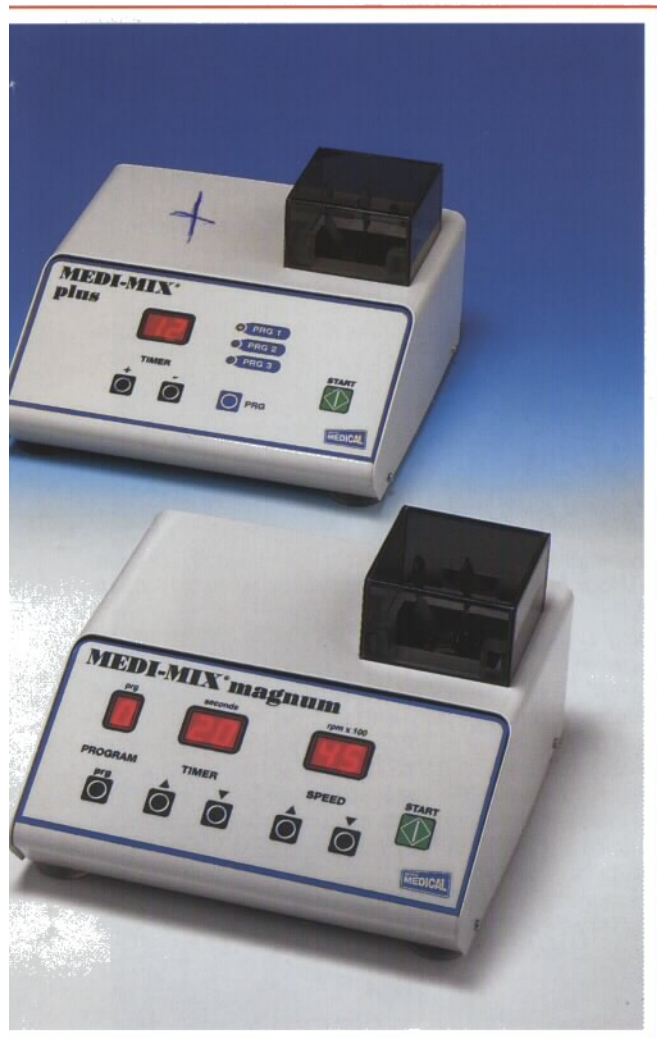
NEW RANGE OF MIXERS FOR AMALGAM IN PREDOSSED CAPSULES

MEDI-MIX plus®

speed: 4500 rpm, fixed
time: adjustable for 0 to 59 seconds
programmes: 3 with free programming
display: visualization of seconds
 led for programme

ART. N° 068-4

MEDI-MIX special®

speed: 4000, 4300 and 4500 rpm adjustable
time: adjustable from 0 to 59 seconds
programmes: 2 with free programming
 with *three-phase** programme
display: visualization of seconds
 led for speed and programme

ART. N° 069-4

MEDI-MIX magnum®

speed: from 3000 to 5000 rpm adjustable
time: adjustable from 0 to 59 seconds
programmes: 9 with free programming
 1 with *three-phase** programme
display: visualization of seconds
 visualization of speed
 visualization of programme

ART. N° 070-4

* **three-phase programme :** phase 1 → mixing for 7 seconds at 4500 rpm
 phase 2 → cooling for 10 seconds
 phase 3 → mixing for 3 seconds at 4000 rpm

According to recent studies carried out by Prof. Jakob Wirz, of the University of Basel, this mixing method improves the physical and chemical features of the amalgam causing a 10-20% increase of resistance to compression, a minor deformation and operating time increased by over 10%.

ALL THE MIXERS ARE EQUIPPED WITH:

- **ELECTRONIC CONTROL** with speed microprocessor
- **SAFETY LID** with interruption of vibrations when opened, in conformity with EEC directive for machines
- **ENTRAP SYSTEM**, an entrapment system for mercury vapour (an exclusivity by Dental Medical)
- **EIGHT-SHAPED MOVEMENT**
- **EXTREMELY LOW NOISE LEVEL**, even at the maximum speed
- **FLEXIBLE** motor and appliance **SUPPORTS**
- also available with **DIFFERENT VOLTAGE AND FREQUENCY** of power supply

All these features make the new series of mixers by
DENTAL MEDICAL:

- ✓ **EXTREMELY PROFESSIONAL**
- ✓ **TECHNICALLY ADVANCED**
- ABSOLUTELY → UP - TO - DATE**

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DENTAL MEDICAL MIXERS, with ENTRAP SYSTEM

DENTAL MEDICAL srl, offers amalgam mixers in caps made with a new and original ecological concept: the MEDI MIX PLUS - SPECIAL - MAGNUM, with ENTRAP SYSTEM (pat. pend.).

After two years of studies and research, the DENTAL MEDICAL srl laboratories, have improved a system capable of holding, in a stable and final way, the mercury vapors which form during the vibration of the amalgam. Vapor forms since mercury evaporates even at room temperature (20°C) and, as a consequence of vibration, the evaporation is enhanced.

It is just at the moment, when the vapors form, that the ENTRAP SYSTEM (trapping system), improved by the DENTAL MEDICAL srl laboratories, comes into action.

The device is made up of a container, holding a particular type of chemical mixture having a high affinity for mercury. It is well known that mercury, is not found free in nature, but only as a compound; sulfide of mercury, on account of its similarity with sulphur, the compound is named cinnabar. The rocks containing cinnabar, a substance which is practically insoluble, (solubility at 18°C=0.00000125%), are heated a 700°C in order for the mercury to produce vapors, which are cooled and condensed. The ENTRAP SYSTEM allows the formation of cinnabar from the vapors formed during vibration, thereby forming an almost insoluble and non volatile compound. The ENTRAP SYSTEM BOX, originally has a straw-yellow color and, as cinnabar forms through saturation of the mercury vapors in its mixture, changes color and becomes first red and finally brown. Saturation is achieved after about 2000 mixings. Once it is reached, the ENTRAP SYSTEM box has to be substituted by removing it without danger of pollution one to the physicochemical stability of cinnabar.

The unit: MEDI MIX PLUS - SPECIAL - MAGNUM, vibrator and ANTRAP SYSTEM is made up of: vibrator, with 8 - shaped movement and electronic timer, lid for holding down and conveying the vapors downwards and the box with the special mixture, placed directly underneath the vibrating body of the mixer.

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ART.

ART.

* **three-phase programme :** phase 1 → mixing for 7 seconds
phase 2 → cooling for 10 seconds
phase 3 → mixing for 3 seconds

According to recent studies carried out by Prof. Jakob Wirz, of the University of Basilea, this mixing method improves the physico-chemical and chemical features of the amalgam causing a 10-20% increase in strength and a reduction in working time of over 10%.

ALL THE MIXERS ARE EQUIPPED WITH:

- ELECTRONIC CONTROL with speed microprocessor
- SAFETY LID with interruption of vibrations when open
- ENTRAP SYSTEM, an entrainment system for metal particles